



HEMOCOLL™


THE ABSORBABLE STERILE FIBRILLAR COLLAGEN WOUND FILLER
FOR DENTAL SURGERY

Hemocoll is NON TOXIC AND NON PYROGENIC



Hemocoll is high purity Type I reconstituted Collagen

Hemocoll to be used in

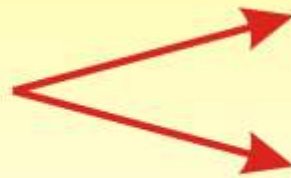

EXTRACTION SITES

BIOPSY SITES

Benefits of Hemocoll:

1. Controls bleeding and stabilizes blood clots (with in 2 to 5 Minutes)
2. Protects wound bed.
3. Helps in regeneration of lost Tissue.
4. Resorbable in 3-4 weeks.
5. Faster healing process.

Hemocoll : Available in the form of dry fibrillar collagen
Individual pack size of 2.5 ml & 5 ml Steriled

Precautions:

- ◆ Hemocoll is not to be used on infected or contaminated wounds.
- ◆ Not intended for use in treating systemic coagulation disorders.
- ◆ Do not store above 32° C (90° F),
- ◆ Discard all open and un used dressing.
- ◆ Do not attempt to resterilize.
- ◆ Safety in pregnancy not established.

Instruction for use : Surface to be treated should first be compressed with dry gauze, remove the gauze carefully, and immediately cover the surface with Hemocoll. The amount required will depend on the force applied and severity of the bleeding. It is necessary to apply moderate pressure over the Hemocoll with a dry gauze.

Mfg Lic. 675

Manufactured & Marketed by



ADVANCED BIOTECH PRODUCTS (P) LTD

77, First Cross Street, Raghavan Colony, Chennai-600 083.

Phone : 91-44-24744650 E-mail : info@advanced-biotech.com website : www.advanced-biotech.com